

# How to send digital Impression



**Office Phone #:** 703.825.7920

**Email:** [crystaldentallab@gmail.com](mailto:crystaldentallab@gmail.com)

**Website:** [www.crystaldentaldesign.com](http://www.crystaldentaldesign.com)

# iTero® How to export/send PLY file

Doctor to Lab

*Practice growth starts with the iTero scanner.*

1) Go to website, [www.myitero.com](http://www.myitero.com)

2) Pick a patient file


- ① Click the file
- ② Click 'Download scan' 
- ③ Click 'PLY (color)'
- ④ Download the file to computer

3) Email PLY file to Crystal

**C O N T A C T** Email: [crystaldentallab@gmail.com](mailto:crystaldentallab@gmail.com) / [cadcam@crystaldentaldesign.com](mailto:cadcam@crystaldentaldesign.com)  
**I N F O R M A T I O N** Website: [www.crystaldentaldesign.com](http://www.crystaldentaldesign.com) Office #: 703.825.7920



1) The doctor will need to sign in at <http://www.mycadent.com> using the credentials provided after purchasing the intraoral scanners.

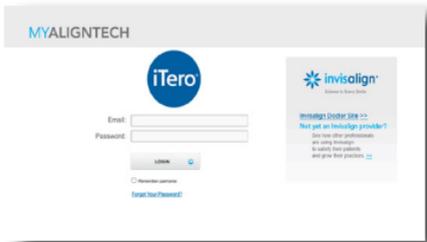

2) The doctor will need to give Crystal Dental Lab's iTero account number (12712) to the iTero Team by calling 800-577-8767.

3) Once the iTero Team approves the account, the doctor can send files to Crystal Dental Lab.

**CONTACT  
INFORMATION**

Email: [crystaldentallab@gmail.com](mailto:crystaldentallab@gmail.com) / [cadcam@crystaldentaldesign.com](mailto:cadcam@crystaldentaldesign.com)

Website: [www.crystaldentaldesign.com](http://www.crystaldentaldesign.com)

Office #: 703.825.7920

# How to send **3M** Scanner file

**Call 1-937-526-8387 and request to  
add Crystal Dental Lab.**

**C O N T A C T**  
**I N F O R M A T I O N**

Email: [crystaldentallab@gmail.com](mailto:crystaldentallab@gmail.com) / [cadcam@crystaldentaldesign.com](mailto:cadcam@crystaldentaldesign.com)

Website: [www.crystaldentaldesign.com](http://www.crystaldentaldesign.com)

Office #: 703.825.7920

# CEREC<sup>®</sup> How to add favorite lab

- 1) Click 'Add', (This will take you to the website, [sironaconnect.com](https://sironaconnect.com).)
- 2) Click 'My favorite laboratories'.
- 3) Click 'Search lab'
- 4) Put zip code, 20121.
- 5) Click 'Start search'.
- 6) Click 'Plus sign to add Crystal Dental Lab'.

<https://www.youtube.com/watch?v=Pub07YMWjrM>

**C O N T A C T** Email: [crystaldentallab@gmail.com](mailto:crystaldentallab@gmail.com) / [cadcam@crystaldentaldesign.com](mailto:cadcam@crystaldentaldesign.com)  
**I N F O R M A T I O N** Website: [www.crystaldentaldesign.com](http://www.crystaldentaldesign.com) Office #: 703.825.7920



- 1) The doctor will need to sign in to the Carestream server at <https://csdentalconnect.com/main.php> or create an account if he/she does not already have one.



- 2) Once signed in, the user will need to search for Crystal Dental Design using the registered email ([crystaldentallab@gmail.com](mailto:crystaldentallab@gmail.com)) and send a partnership request.
- 3) Once the request is sent, Crystal Dental Design will review and accept the request.


Additional information available at [https://help.csdentalconnect.com/en\\_US/](https://help.csdentalconnect.com/en_US/)

<b>C O N T A C T I N F O R M A T I O N</b>	Email: <a href="mailto:crystaldentallab@gmail.com">crystaldentallab@gmail.com</a> / <a href="mailto:cadcam@crystaldentaldesign.com">cadcam@crystaldentaldesign.com</a>
	Website: <a href="http://www.crystaldentaldesign.com">www.crystaldentaldesign.com</a> Office #: 703.825.7920



1) The doctor will need a 3Shape account and portal.

Go to <https://partner.3shape.com> to sign in or create an account

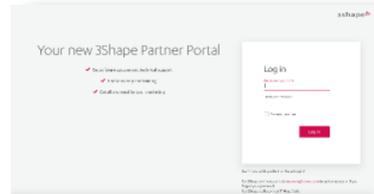

2) Once logged in, the user can find and add Crystal Dental Design by entering the registered email (crystaldentallab@gmail.com)

3) Crystal Dental Design will be able to receive patient files via the 3Shape Partner Portal.

**C O N T A C T** Email: [crystaldentallab@gmail.com](mailto:crystaldentallab@gmail.com) / [cadcam@crystaldentaldesign.com](mailto:cadcam@crystaldentaldesign.com)  
**I N F O R M A T I O N** Website: [www.crystaldentaldesign.com](http://www.crystaldentaldesign.com) Office #: 703.825.7920